


Patient Instructions for the GENUAIR[®] Inhaler

The GENUAIR[®] device uses a feedback mechanism to deliver the correct preparation and inhalation of the loaded dose to patients.

Please refer to the patient information leaflet enclosed with your product package for complete instructions and important safety information.



1 Prepare your dose

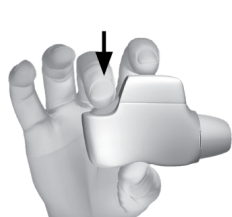

Green
& Go

Step 1: Hold the inhaler horizontally, with the coloured button on top. Do not tilt the inhaler. Press and RELEASE the button and breathe out completely, away from the inhaler. The control window will change from **red** to **green**. Remember to release the button so the inhaler will work properly. If the control window is still **red**, repeat these steps until it changes to **green**.

2 Inhale strongly and deeply



Red
& Done

Step 2: Place your lips tightly around the mouthpiece and inhale **STRONGLY** and **DEEPLY** through your mouth.

- While you breathe in you will hear a “CLICK” which signals that you are using the GENUAIR[®] inhaler correctly.
- Keep breathing in even after you have heard the inhaler “CLICK” to be sure you get the full dose.
- Remove the GENUAIR[®] inhaler from your mouth and hold your breath for as long as is comfortable, then slowly breathe out away from the inhaler.
- Check the colour of the control window. If the coloured control window is still **green**, please repeat inhaling strongly and deeply through the mouthpiece (see Step 2). If the window still does not change to **red**, you may have forgotten to release the button before inhaling or may not have inhaled correctly. If that happens, try again.
- Some patients may experience a grainy sensation in their mouth or a slightly sweet or bitter taste.
- Do not take an extra dose even if you do not taste or feel anything after inhaling.

Please contact your doctor or pharmacist if the control window is still green after repeated attempts.